

Provider setup

Philips Respironics Sleep Therapy System

Step 1: Set up device

To access provider screen:

1. In patient screen, highlight **Setup**.
2. Press and hold ramp button and control wheel simultaneously for 4-5 seconds until beep.
3. Provider screen then appears. Highlight **Setup** again and press control wheel.

To program device:


1. Turn control wheel to specific setting and press.
2. Turn control wheel to desired option and press control wheel to lock.
3. Turn control wheel to next setting and repeat.
4. To exit, turn control wheel to **Back** and press.

Menu options will vary based on device model being used. Refer to provider manual for complete description of each setting and option.



Note: the screen will automatically reset to patient mode after 60 seconds of inactivity.

Setting	Option
Back	Auto CPAP
Mode	HPa/cmH2O BiPa cmH2O
Auto max	(auto min)-20.0
Auto min	4.0 (auto max)
Flex type	None C-Flex A-Flex
Flex	1 2 3
Flex lock	on off
SYSTEM ONE resistance	0 X1 X2 X3 X4 X5
Lock SYSTEM ONE	on off
Ramp time	0:00-0:45
Ramp start	4.0 (auto min)
Humidifier	0 1 2 3 4 5
Auto on	on off
Auto off	on off
Mask alert	on off
LED backlight	on off
Show AHI/leak/PE	on off
Split night	on off
Split night start	
GMT offset	-12 -12
ENGLISH	
Back	

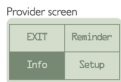

Step 2: Verify setup

1. Turn control wheel to highlight **Exit** and press. You are now in patient screen.
2. Turn control wheel to **Therapy** and press.
3. Pressure will be displayed and device will start. (Auto CPAP will display min pressure)



Step 3: Set reminders

Return device to provider screen, then:

1. Turn control wheel to **Reminder** and press.
2. Turn control wheel to desired option and press.
3. Turn control wheel to **Back** and press.



Back	Reminder off	30	90
	180	270	365

Step 4: View information

In provider screen:

1. Turn control wheel to **Info** and press.
2. Turn control wheel to desired option and press to view information as identified.
3. To exit out of any option, turn control wheel to **Back** and press.

Option	Data display
Back	Therapy hours 7 day avg: 0:00
Phone-in	AHI 7 day avg: 0.0
Compliance VIC	30 day avg: 0:00
Therapy hours	Back
Blower hours	Blower hours 7 day avg: 0:00
Days=4	Total hours: 0:00
System leak	Back
AHI	Periodic breathing 7 day avg: 0%
Periodic breathing	7 day avg: 0%
Reset data	30 day avg: 0%
Machine hours	System leak 7 day avg: 0%
Back	30 day avg: 0%
	Machine hours Total hours: 0.0
	Back

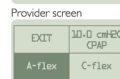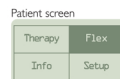

Note: Flex options vary among models.

Step 5: Flex demo

1. From the patient screen, highlight **Flex**.
2. Press and hold ramp button and control wheel simultaneously for 4 to 5 seconds until beep. You are now in provider screen.
3. New screen appears with **CPAP** and **Flex** modes in each quadrant.
4. Turn control wheel to the specific mode you want to demo and press.
5. Device will auto start blower and deliver the selected mode. The screen will automatically cycle to display the waveform.
6. After 10 minutes of no action, device resets to patient screen and blower turns off.

Step 6: Undocking

Refer to diagram below. Hold entire unit at each end and slide release latch to the right.

